SURGENT™ SINUS IRRIGATION CATHETER

Directions for Use

CAREFULLY READ ALL INSTRUCTIONS PRIOR TO USE

PACKAGING

STERILE: Sterilized with ethylene oxide gas. Do not use if package is damaged.

STORAGE: Store in a cool, dry place.

SINGLE USE: Single patient use only, do not reuse.


INTENDED USE

The device is intended for the removal of mucus, biofilms and debris from the paranasal sinuses using irrigation in patients with chronic rhinosinusitis.

WARNINGS

- Do not use if package is damaged.
- Single patient use only.
- Endoscopic visualization during use is required.
- Use in the sphenoid sinus may result in impingement of the internal carotid artery or optic nerve.
- Prior CT is recommended to evaluate bone integrity in order to reduce risk of fracture.
- Pre-bending is recommended to steer the irrigation catheter to the desired location
- Use only liquid for balloon inflation.
- Excessive inflation may increase risk of adjacent bone fracture, nerve damage, vascular damage, or soft tissue damage.
- Do not inflate balloon in ostium.
- Do not remove device from patient while balloon is inflated.
- Do not irrigate sinus if the balloon fully occludes sinus ostium.
- Sinus contents may flow back through irrigation port if no syringe is attached to irrigation port.
- To prevent inadvertent fluid extravasation into adjacent compartments, do not use this device in the presence of a bony dehiscence or defect in any sinus wall.

DEVICE OVERVIEW

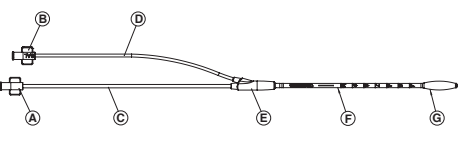A. Irrigation Port (central lumen)

B. Balloon inflation port

C. Irrigation Line

D. Inflation Line

E. Hub

F. Malleable Shaft

G. Balloon

INSTRUCTIONS FOR USE

- Endoscopically evaluate sinus cavity for suitability of balloon placement and inflation.
- Pre-bend catheter to facilitate insertion into desired sinus.
- Attach balloon fill syringe to balloon inflation port B.
- Pre-inflate balloon with liquid, then deflate balloon and residual air. Leave syringe attached.
- Attach pre-filled irrigation syringe to irrigation port A.
- Insert balloon catheter into sinus cavity.
- Slowly inflate balloon to desired volume.
- Partially deflate balloon to enable irrigant to exit sinus.
- Irrigate thoroughly while observing evacuation of sinus debris.
- Repeat inflation and irrigation sequence as desired.
- Deflate balloon and remove device carefully.

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© 2017 Ondine Biomedical Inc.
Ondine Biomedical Inc. 888-1100 Melville Street Vancouver, BC, Canada V6E 4A6