

Clear your nose of germs in 2 simple, pain-free steps



Germs in your nose can cause infection in open wounds and incision sites. That's why we use the MRSAid™ Photodisinfection System to get rid of the germs in your nose before your procedure, significantly reducing your risk of infection.

How does MRSAid work?

First, we swab the inside of your nose with a special blue liquid that attaches to the germs.

Then, we insert small nasal tips and turn on the red light.



The red light causes the blue liquid to undergo a chemical reaction that kills the germs in your nose but does not hurt the surrounding tissue. The treatment is complete in about 5 minutes.

What you may feel

During the swabbing, it may tickle the inside of your nose or cause you to feel the need to sneeze. You may also feel a sense of warmth or notice different smells when the light is in your nose. This is normal.

If you experience pain or discomfort as a result of the treatment, please notify your nurse or caregiver immediately. Your safety is our number one concern.